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
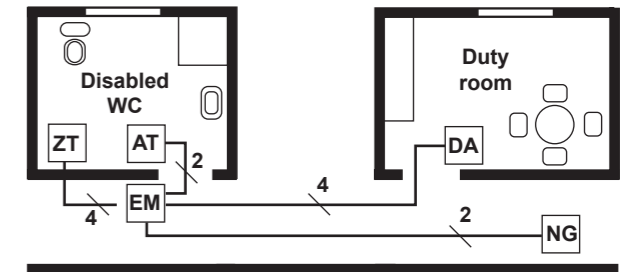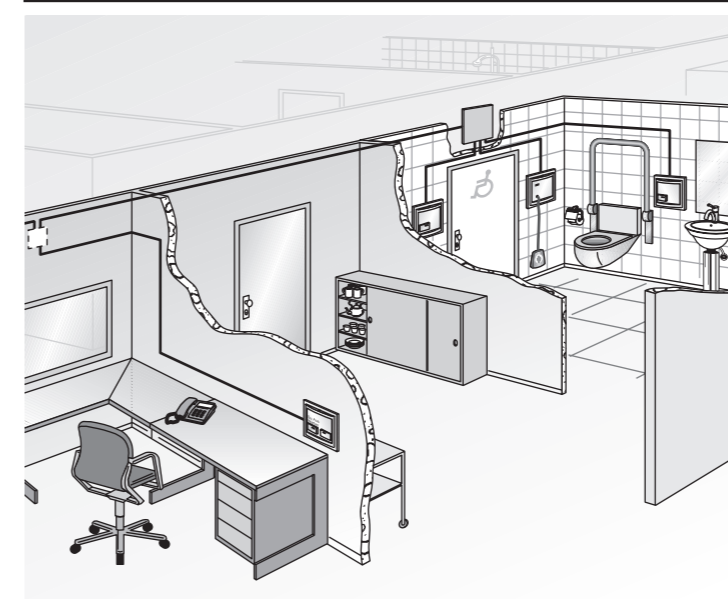
- Legend
 ZT = Call switch for call release
 AT = Cancel push-button
 EM = Electronic module incl. LED lamps
 DA = Duty room unit
 NG = Power supply

Example of application: System compact CL in a WC for the disabled (with auxiliary accessories)

System compact CL

Order No. 76910D1

Description of functions:

Call release: A call released by a disabled person by means of the call switch (ZT) is acknowledged through the lighting up of the reassurance lamp (integrated in the ZT) and is displayed on the corridor indicator lamp (EM) installed outside the WC. Note: Optionally, a parallel lamp on the EM can be connected.

Duty room: The duty room unit (DA) installed in the duty room indicates the call to the personnel optically by illuminating a red call lamp and acoustically by means of a buzzer sound. The call can be acknowledged with the SU-AUS (mute) button, i.e. the acoustic message is suppressed for a period (approx. 15-20 sec.).

Cancellation: The released call is cancelled by pressing the cancel push-button (AT) in the WC.

Note: Optical and acoustic messages as continuous signal. The potential-free contact is set either with the call frequency or for the call period (see configuration). Installation in equipment or equipment connection boxes according to DIN 49073 (min. 60 mm depth).

Assembly: 1 off call switch with red LED reassurance lamp and 2 m perlons cord incl. cover plate, cover frame.

1 off cancel push-button incl. cover plate with green cancel push-button, cover frame.

1 off electronic module with integrated red LED lamp and electronics, cover plate, cover frame.

1 off duty room unit with red call lamp and buzzer incl. cover plate with white SU-AUS (mute) button, cover frame.

1 off power supply for the low-voltage supply of the equipment. (It should be bolted firmly to the respective surface using its screw holes.)

Input voltage, power supply	230V / 50 Hz
Output voltage, power supply	24V DC / 12W / 0.5A
Max. operating temperature, power supply	50°C
Length x width x height	86 x 41 x 22
Cover plates and cover frames	RAL 9016
1 x sealing kit each for ZT and AT	IP54